FD 312 wet wipes

DE	Flächen- desinfektion	PL	Dezynfekcja powierzchni
EN	Surface Disinfection	CS	Dezinfekce ploch
FR	Désinfection des surfaces	SK	Dezinfekcia plôch
IT	Disinfezione di superfici	SL	Površinska dezinfekcija
ES	Desinfección de superficies	HR	Dezinfekcija površina
PT	Desinfecção de superfícies	HU	Felületi fertőtlenítés
NL	Oppervlakte- desinfectie	EE	Pindade desinfitseerija
SV	Yt-desinfektion	LV	Virsmu dezinfekcijas lõrdzекlis
FI	Pintojen desinfiointiaine	LT	Paviršių dezinfekcija
NO	Overflate- desinfeksjon	RU	Средство для деинфекции поверхностей
DA	Overflade- desinfektion	BG	Дезинфектант за повърхности
EL	Απολύμανση επιφανειών	RO	Dezinfecția suprafețelor
		TR	Yüzey Dezenfeksiyonu

DÜRR DENTAL AG
Höpfigheimer Straße 17
74321 Bietigheim-Bissingen, Germany
Tel.: +49 7142 705-0
Fax: +49 7142 705-500
info@duerr.de · www.duerr.de

 DÜRR
DENTAL

Z.119-2015-01

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FD 312 wet wipes

Surface Disinfection

FD 312 wet wipes are ready-for-use cloths for the wipe disinfection and cleaning of floors, practice inventory etc.

Composition: Wet wipes contain ready-for-use FD 312 solution. 100 g solution of active substance contain 65 mg alkyl-benzyl-dimethyl-ammonium chloride, non-ionic surfactants, alkaline cleaning components, complexing agents and additives in aqueous solution.

Effective spectrum: Bactericidal, tuberculocidal, fungicidal¹⁾, limited virucidal activity²⁾ (enveloped virus, incl. HBV, HCV and HIV). **List of VAH/DGHM.** Tested in accordance with EN 13727, EN 13624.

¹⁾ Testing with *Candida albicans*.

²⁾ In accordance with the recommendations of the RKI (Robert Koch Institute) (Bundesgesundheitsbl. 47, 62-66, 2004).

Application: Wet the surface and allow the product to remain in contact for 15 min. One cloth is sufficient for 6 – 8 m².

Close the storage pack properly after removing each cloth.

Note: Store in a cool, dry place in the original pack. Dispose of large quantities in accordance with official regulations, if necessary dispose of as problem waste. Packaging should be recycled. Dispose of used cloths in the household waste – do not throw them down the toilet.

LOT  see imprint.

Wear protective gloves. **IF IN EYES:** Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **IF ON SKIN:** Wash with plenty of water.

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Distribuitor în România: DÜRR DENTAL AG
Türkiye'de satış: DÜRR DENTAL AG