# Orotol<sup>®</sup> ultra

- DE** Sauganlagen-Desinfektion  
**Achtung**
- EN** Disinfection of suction systems  
**Warning**
- FR** Désinfection des systèmes d'aspiration  
**Attention**
- IT** Disinfettante per impianti d'aspirazione  
**Attenzione**
- ES** Desinfección de los sistemas de aspiración  
**Atención**
- PT** Desinfecção de sistemas de sucção  
**Atenção**
- NL** Afzuigsystemen-desinfectie  
**Waarschuwing**
- SV** Suganläggningars-desinfektion  
**Varning**
- FI** Imulaitteiden desinfiointiaine  
**Varoitus**
- NO** Sugeanlegg-desinfeksjon  
**Advarsel**
- DA** Sugeanlægsdesinfektion  
**Advarsel**
- EL** Απολύμανση συστημάτων αναρρόφησης  
**Προσοχή**
- PL** Dezynfekcja systemu ssącego  
**Uwaga**
- CS** Dezinfekce odsávacích zařízení  
**Varování**
- SK** Dezinfekcia odsávacích zariadení  
**Pozor**
- SL** Dezinfekcija sesalnih naprav  
**Pozor**
- HR** Dezinfekcija usisnog sustava  
**Upozorenje**
- HU** Elszívó berendezések fertőtlenítése  
**Figyelem**
- EE** Imurisüsteemide desinfektsioon  
**Hoiatus**
- LV** Atsūkšanas sistēmu dezinfekcija  
**Uzmanību**
- LT** Siurbimo įrangos dezinfekcija  
**Atsargiai**
- UA** дезінфекція аспіраційних установок  
**Обережно**
- BG** Дезинфекция на аспиратори  
**Внимание**
- RO** Dezinfecția sistemelor de aspirație  
**Atenție**
- TR** Aspirasyon ünitelerinin dezenfeksiyonu  
**Dikkat**

CE 0124

MD

duerrodent GmbH + Co. KG  
Max-Planck-Straße 27  
70806 Kornwestheim, DE  
a DÜRR DENTAL company

Tel. +49 7142 705-0  
info@duerrdental.com  
www.duerrdental.com  
Made in Germany

**DÜRR  
DENTAL**

<b>Anwendung / Application</b>	<b>Konzentration – Zeitrelation / concentration – time relation</b>	
<b>Anwendungsempfehlung zur Sauganlagendesinfektion nach VAH / Recommended application for disinfection of suction systems according to VAH</b>	<b>Bakterizid / Levurozid Bactericidal / Yeasticidal</b>	1 % – 5 min.
	<b>Tuberkulozid / Tuberculozidal /</b>	4 % – 30 min.
<b>Wirkungsspektrum / Type of activity</b>	<b>Geprüft gemäß / Tested according to</b>	<b>Konzentration – Zeitrelation / concentration – time relation</b>
<b>Bakterien und Hefen / Bacteria and yeasts</b>		
<b>Bakterizid / Levurozid / Bactericidal / Yeasticidal</b>	EN 13727, EN 14561, EN 13624, EN 14562	1 % – 5 min.
<b>Tuberkulozid / Tuberculozidal</b>	EN 14348, EN 14563	4 % – 30 min. 2 % – 60 min.
<b>Mykobaterizid / Mycobactericidal</b>	EN 14348, EN 14563 <sup>1)</sup>	4 % – 60 min.
<b>Schimmelpilze / Moulds</b>		
<b>Fungizid / Fungicidal</b>	EN 13624, EN 14562	4 % – 4 h
<b>Sporen / Spores</b>		
<b>Sporizid (Clostridium difficile) / Sporicidal (Clostridium difficile)</b>	EN 17126	1 % – 30 min.
<b>Viren / Viruses</b>		
<b>Begrenzt viruzid (behüllte Viren inkl. HBV, HCV, HIV sowie Coronaviren) / Virucidal activity against enveloped viruses (enveloped viruses incl. HBV, HCV, HIV as well as Coronaviruses)</b>	EN 14476, EN 17111	1 % – 5 min.
	DVV/RKI Leitlinie / DVV/RKI-Guideline	1 % – 15 min.
<b>Viruzid (behüllte und unbehüllte Viren) / Virucidal (enveloped and non-enveloped viruses)</b>	EN 14476, EN 17111, DVV/RKI Leitlinie / DVV/RKI-Guideline	2 % – 60 min.
<b>Adenoviren / Adenoviruses</b>	EN 14476, EN 17111, DVV/RKI Leitlinie / DVV/RKI-Guideline	1 % – 5 min.
<b>Noroviren / Noroviruses</b>	EN 14476, EN 17111, DVV/RKI Leitlinie / DVV/RKI-Guideline	1 % – 5 min.
<b>Polyomaviren SV40 / Polyomaviruses SV40</b>	EN 14476, EN 17111, DVV/RKI Leitlinie / DVV/RKI-Guideline	1 % – 5 min.
<b>Polioviren / Polioviruses</b>	EN 14476, DVV/RKI Leitlinie / DVV/RKI-Guideline	2 % – 60 min.

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## Orotol® ultra

### Disinfection of suction systems

Orotol® ultra is a highly effective preparation for the cleaning, disinfection, deodorisation and care of all dental suction units as well as spittoons, and is suitable for all amalgam separators. Select disinfectant and cleaning components achieve intensive, foam-free care while being gentle on materials.

**Composition:** 100 g Orotol® ultra contains 25 g sodium carbonate peroxyhydrate, 5-15 g alkaline cleansing agents, < 5 g complexing agents and < 5 g special antifoaming agents, fragrances and excipients.

**Spectrum of activity:** bactericidal, tuberculocidal, mycobactericidal, fungicidal, sporicidal (*clostridium difficile*), virucidal (enveloped viruses incl. HBV, HCV, HIV and coronaviruses as well as non-enveloped viruses such as adenoviruses, noroviruses, polioviruses, SV40 polyomaviruses). Orotol® ultra is included in the **VAH list and IHO list of disinfectants**. Tested under high load in accordance with EN 13727, EN 14561, EN 13624, EN 14562, EN 14348, EN 14563, EN 17126, EN 14476, EN 17111 as well as the DVV/RKI guideline with and without load.

**Application:** Use Orotol® ultra regularly once or twice a day, e.g., before lunch and after the end of treatment. Once-daily application should be done exclusively at the end of the day. Transfer Orotol® ultra from the refill pack into the storage container. Apply Orotol® ultra in a 1%, 2% or 4% concentration. For this purpose dissolve 20, 40 or 80 g Orotol® ultra (equivalent to 2, 4 or 8 dosing spoons) in 2 L of lukewarm water (approx. 30 °C) and evacuate 1 L after a waiting time of 15 minutes using the OroCup for example. Add the remaining solution to the spittoon. In accordance with the VAH the required concentration time relation is 1% - 5 min. (bactericidal / yeasticidal) or 4% - 30 min. (tuberculocidal). For mycobactericidal efficacy, 4% - 60 min. is required. Fungicidal efficacy is achieved at 4% - 4 hours and sporicidal efficacy against *clostridium difficile* is achieved at 1% - 30 min. Inactivation of enveloped viruses takes place at 1% - 5 min to 15 min and non-enveloped viruses at 1 to 2% - 5 to 60 min (see table). Prepare fresh working solution for each application. Follow the device manufacturer's instructions and country-specific requirements. For further information please also see product information on the internet. Note: Do not mix Orotol® ultra with other products. Thoroughly flush the suction systems with at least 1 L of water before and after applications of Orotol® ultra. After surgical procedures, aspirate at least one glass of cold water in spurts. Do not introduce foaming agents into the suction system or the spittoon. Changes in the colour and odour of Orotol® ultra during storage have no impact on efficacy. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

**LOT**  see imprint on refill pack.

Warning. Causes serious eye irritation. Keep cool. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves and eye/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Protect from sunlight. Dispose of contents/container to hazardous or special waste collection point.

UK-REP: DÜRR DENTAL (Products) UK Ltd., 14 Linnell Way - Telford Way Industrial Estate, Kettering, Northants NN16 8PS, United Kingdom, info@duerruk.com

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orochemie GmbH + Co. KG