Orotol[®] ultra

Sauganlagen-Desinfektion

Disinfection of suction systems

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Orotol[®] ultra is a highly effective preparation for the simultaneous **disinfection, deodorization, cleaning and care** of all dental suction systems as well as spittoon bowls, and is suitable for all amalgam separators.

Composition: 100 g Orotol[®] ultra contain 25 g sodium carbonate peroxyhydrate, 5 - 15 g alkaline cleansing agents, < 5 g complexing agents and < 5 g special antifoaming agents as well as fragrances and adjuvants.

Effective spectrum: Bactericidal, incl. for Tb, sporicidal (Clostridium difficile), fungicidal, virucidal¹⁾ (encapsulated viruses such as vaccinia viruses, incl. HBV, HCV, HIV as well as non-encapsulated viruses, such as adenoviruses, polyomaviruses SV 40, polioviruses, noroviruses). Registered for **VAH/DGHH certification**. Tested in accordance with RKI/DVV guideline.

Tested in accordance with EN 13704, EN 13727, EN 13624, EN 14348, EN 14476, EN 14561, EN 14562, EN 14563.

¹⁾ In accordance with RKI recommendation (Federal Health Bulletin 47, 62-66, 2004).

Application: Transfer Orotol[®] ultra from the refill pack into the storage tin. Apply Orotol[®] ultra in a 1% concentration. For this purpose dissolve 10 g Orotol[®] ultra (equivalent to 1 dosing spoon) in 1 litre lukewarm water (approx. 30 °C) and aspirate with the OroCup. In accordance with VAH/DGHH, the exposure time is 5 min., while for tuberculosis bacteria it is 4% and 60 min. For inactivation of viruses we recommend 2% and 60 min. (see table in DE).

We recommend disinfecting and simultaneously cleaning the suction system with Orotol[®] ultra once or twice a day, depending on the workload. If two applications are needed, do so before lunchtime and after the end of treatment. If only one application is needed, do so exclusively in the evening after the end of treatment. Disinfect and clean spittoon bowl also with at least 250 ml solution. Prepare a fresh ready-to-use solution for each application. Rinse disinfected system thoroughly with water after the exposure time has elapsed. If the tap water is problematic (e.g. high water hardness) or if sparingly soluble residues of prophylaxis powders or pearl products have to be dissolved, we recommend using MD 555 special cleaner in addition to Orotol[®] ultra.

Notes: After surgical procedures, aspirate one glass of cold water in spurts. **Do not** introduce foaming agents into the suction system or the spittoon bowl.

  see imprint on refill pack.

Warning. Causes serious eye irritation. Keep cool. Protect from sunlight. Avoid breathing dust. Wear protective gloves and eye/face protection. If IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

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DÜRR DENTAL AG