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EN

Characteristics
Concentrated hygienic and cleaning solution. Refreshes and deodorizes environments; cleans quickly and thoroughly. Does not corrode rubber, wood, plastic materials, porcelain, ceramics, enamel. Is not aggressive on metals.
Uses
Suitable for: - cleaning dental surgeries and laboratories (washbasins, tiles, floors, bathrooms, doors); - cleaning wards, corridors, kitchens and rooms in hospitals and clinics.
Instructions for use
Dilute 2-3 dosage caps in 5 litres of water. For cleaning ingrained dirt, use a few drops of the concentrated solution on a damp cloth. When diluted gives 30-50 litres of solution.
Warnings and storage conditions
Keep the closed container in a cool dry place and out of the reach of children. Avoid releasing the container to the environment after use. Biodegradable product more than 90% (Art. 2 and 4 low 26/04/83 N°136).
Cleaner for hard surfaces
Ingredients compliant with regulation (EC) no. 648/2004 less than 5% cationic surfactants, non-ionic sufactants, EDTA salt Aroma, Limonene Preservative, BHT.

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EN
Properties
Concentrated cleaning solution for disinfection and deodorization. Refreshes and deodorizes the environment; cleans quickly and thoroughly. Does not attack rubber, wood, plastic materials, porcelain, ceramics and enamel. Does not corrode metal.
Applications
Use for:
- dental clinics and dental laboratories (sinks, tiles, floors, bathrooms, doors);
- hospital and clinic departments, corridors, kitchens and other areas.
Use
Use in the following cases:
- disinfection of dental treatment rooms and dental technology laboratories (sinks, tiles, floors, toilets, doors).
- disinfection of all departments, corridors, kitchens and other areas of the hospital and clinic.
Use
Measure 2-3 cups of product in 5 liters of water.
For stubborn stains, use a cloth dipped in a few drops of the concentrate.
Obtain 30-50 liters of diluted solution.
Precautions and storage conditions
After use, close the packaging carefully and store the product in a place inaccessible to children.
Do not dispose of the container in the environment after use.
The product is biodegradable by more than 90% (Art. 2 and 4, Italian Law No. 136 of 26/04/83)
Hard surface cleaner
Complies with CE 648/2004
<5% cationic surfactants, non-ionic surfactants, EDTA salt fragrance: Limonene, BHT

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
HIGH PERFORMANCE CLEANING SOLUTION
FOR WASHABLE SURFACES

CONCENTRATED

USE AT 1% - 3%

Last Update: 04/2015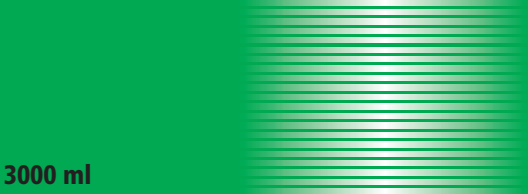

Reorder N. [REF](#) C810037

U11886815/04

Zhermack S.p.A. | via Bovazecchino, 100
45021 Badia Polesine (Rovigo) Italy
T +39 0425 597611 | www.zhermack.com