# ULTRASUC SUPER CLEANER

Stand:231017

**REF** 000 779

**Inhalt/Content/Contenu/ Zawartość/ Obsah/ Tartalma:**  
**e 2 Liter/litres/litry/ litry/liter**

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Ⓧ Low-foaming cleaning concentrate for cleaning dental suction systems

Low-foaming cleaning concentrate for cleaning dental suction systems, including their drainage lines. Scaling and clogging are avoided with regular use and existing deposits are dissolved.

**Indications of use:** In problem cases, the special cleaner is used as a 10% solution (100 ml / l water). The exposure time is 1-2 hours. For regular use, the cleaner is used weekly as a 5% solution (50 ml / l water) with a 30 min exposure time. 2 litres of the prepared solution are drawn through the suction hoses and after the exposure time the system is rinsed with a litre of water. (This corresponds to 2 x 1/2 dosing caps in 2 litres water).

**Safety notes: H319** causes severe eye irritation. Wash thoroughly after handling. Wear protective gloves and eye protection. **IN CASE OF CONTACT WITH THE EYES:** Rinse out carefully with water for several minutes. Remove any contact lenses if easily possible. Continue to rinse out. **If eye irritation persists:** Get medical advice/attention. Ingredients 648/2004/EC: <5% amphoteric tensides, <5% phosphonates.

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Müller & Weygandt GmbH • Reichardsweide 40 • D-63654 Büdingen • Tel.: +49 (0)6042 – 88 00 88