## UltraPro™ Tx Air Hygiene Handpiece

EN

### Description:

UltraPro™ Tx Air Hygiene Handpiece is a light-weight, ergonomic, friction grip handpiece. The motor has a working output of 4000 RPM at variable working pressures of 35–45psi.

### Indications for Use:

UltraPro™ Tx handpieces are intended for prophylaxis procedures, for use by a trained professional in dentistry.

### Instructions:


1. Prior to attaching air supply, bleed air line of moisture and contamination.
2. Attach handpiece (motor and nosecone) to air hose.
3. Confirm working pressure is between 35–45psi.
4. Attach prophyl angle by pushing it until it is fully seated. Friction grip will hold the angle in place.
5. To remove the prophyl angle, grasp the nose cone and firmly pull the prophyl angle in a straight outward direction.
6. For a smoother swivel between the motor and nose cone, the "O" ring may be removed from the motor stem.

### Sterilization and Maintenance of Motor and Nosecone:


1. Remove prophyl angle from the handpiece.
2. Disconnect handpiece from hose.
3. Separate the motor section and the nosecone.

Sterilization Processing Instructions			
Initial treatment at the time of use:	Sterilization is required before every use.		
Preparation before Sterilization	Wipe device with 70% IPA soaked gauze.		
Maintenance, inspection and testing	Visually inspect that the device is not bent or damaged and free of debris. If debris is present remove before sterilizing.		
Packaging	Place selected device in approved sterilization packaging (ISO11407 & ISO11607).		
Sterilization	Place sterilization package in autoclave using the following settings:		
	Cycle Type	Minimum Sterilization Exposure Time	Minimum Sterilization Exposure Temperature
	Pre-vacuum	3 mins	132°C / 270°F
	Gravity	30 mins	132°C / 270°F
			Minimum Dry Time
			20 mins
			30 mins
	Do not overload the sterilizer.		
Storage	Keep device in sterilized packaging until ready for use. Keep out of sunlight and moisture.		
Additional information	If sterilization packaging is compromised, repeat sterilization process.		
	This procedure was validated by an independent and accredited laboratory.		

4. After sterilizing, place two drops of handpiece oil into the motor air tube (Fig. 1) and 1–2 drops of oil into both ends of the nosecone (Fig. 2). To properly dispense the oil, run the motor for several seconds.
5. Apply one drop of handpiece oil to the "O" ring one time per week.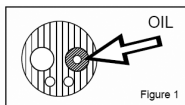

Lubricating Motor

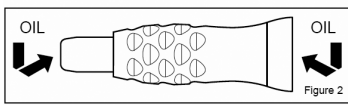Lubricating Nosecone

### Precautions and Warnings:

1. Carefully read and understand all instructions before using.
2. Never place handpiece in ultrasonic cleaner.
3. Do not use sterilization bags containing disinfectants.
4. Always check that the compressed air is dry and free of oil.
5. Do not exceed 45psi when operating handpiece.
6. For best results, use prophyl angles with 2.30mm – 2.35mm shanks.
7. Always check that the angle is firmly seated in place on the handpiece before beginning the procedure.
8. For professional use only.

### Warranty:

Ultradent Products, Inc.'s ("Ultradent") UltraPro™ Tx Air hygiene handpiece warranty obligations are limited to the terms set forth below and are not transferable: Ultradent warrants this UltraPro™ Tx Air handpiece, including the motor and nosecone, against defects in materials and workmanship under normal use and proper maintenance for a period of ONE (1) YEAR from the date of purchase by the first retail purchaser ("Warranty Period"), as evidenced by invoice and shipping records. If a defect arises and a valid claim is received within the Warranty Period, Ultradent will, at its option repair or replace the defective product. The new Warranty Period for the repaired or replaced product shall be either ninety (90) days from the date of replacement or repair of the defective product or the amount of time remaining on the original Warranty Period, whichever period of time is greater. When a refund is given, the product for which the refund is provided will be returned to Ultradent. If handpiece is modified, misused or disassembled, warranty is void.

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© Copyright 2019 Ultradent Products, Inc. All Rights Reserved.