# Big-Etch

Stand 1423

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# Big-Etch

Stand 1423

## Information and usage instructions

Contents: 50 ml (62 g)  
Supply syringe  
10 Applicator syringes  
50 Applicator tips  
in a practical pack

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Müller & Weygandt GmbH · Industriestrasse 25  
D-63654 Büdingen · Tel.: +49 (0) 60 42 – 88 00 88

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### General:

M+W etch gel is a 36% phosphoric acid etch gel with optimum thixotropic flow properties. It is used for the etching of tooth substance as part of the melt-etch technique and the total-etch technique. If used in accordance with the safety and usage instructions, the product has no known side effects. Underfilling materials may be etched by the etch gel in case of longer acting times.

M+W etch-gel can be easily decanted and reliably applied, does not dry out, and can be stored for at least 2 years following the production date at a maximum storage temperature of 25 °C.

- For dental use only!
- Do not use after the expiry date.
- Seal syringe immediately after use.

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### Safety instructions

**H 314** Causes severe skin burns and severe eye damage. Wear protective gloves / eye protection. **IF IN EYES:** Rinse carefully with water for several minutes. Remove contact lenses if possible. Continue rinsing. Call the **POISON INFORMATION CENTRE** or a doctor immediately. **IF SWALLOWED:** rinse mouth. **DO NOT** induce vomiting. **IF IN CONTACT WITH SKIN:** wash with copious amounts of water and soap. Store in a closed container. Disposal of contents / container in accordance with local / regional / national/international regulations. Contains ortho phosphoric acid.

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CE 0297

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### Filling the applicator syringe:

An empty applicator syringe is locked onto the supply syringe (by turning 180° clockwise). The small syringe is filled by carefully and slowly pressing the large syringe piston. If necessary, also simultaneously turn the small syringe piston slightly, and ensure that the small syringe is not overfilled or the piston pressed out of the syringe! Afterwards please detached the applicator syringe from the Big Etch Syringe and sealing the Big Etch Syringe. A tube is attached to the applicator syringe.

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#### **Use: Preparation**

The areas of the tooth substance to be etched must be clearly defined.

1.) The use of a dental dam is recommended! Avoid all contact with soft tissue (skin, oral mucosa, eyes etc.). If the etch-gel comes into contact with soft tissue, rinse immediately with plenty of water and consult a doctor if necessary.

2.) Before preparation, clean the tooth using a rubber cup with fluoride- and fat-free polishing paste.

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3.) Remove cleaning residues carefully with water and clean and dry the tooth surface with oil-free compressed air.

4.) Prepare the cavity accordingly, chamfer the melt edges in order to produce permanent filling edges. Rinse out the cavity thoroughly with water and dry with oil-free compressed air. Avoid excessive drying!

5.) Protect the pulp in deep cavities (e.g. cover with a thin coating of a suitable underfilling material). The cavity should be dry and free of impurities, and excessive drying of the dentine should be avoided.

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8.) Dry with oil- and water-free compressed air. The etched surface should not be touched or contaminated by saliva until continuation of the treatment. If contamination occurs after etching and drying, etching must be repeated briefly (approx. 10 sec.).

#### **The etched enamel should appear chalky white!**

After etching, the use of adhesives etc. can be continued in accordance with the manufacturer's usage information.

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