# Mundspülbeckenreiniger AF

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## **GB** Cuspidor cleaner AF Content: e 750 ml

**Type of application:** According to MPG (German Medical Devices Act): The cuspidor cleaner AF is a special alkali formulation for cleaning, disinfecting and care of spittoons in dental treatment units. The cuspidor cleaner AF, reliably removes depositions of lime scale blood and dye residues from plaque tablets. According to the Biocidal Products Directive: The disinfectant can also be used to disinfect and clean wash basins, WCs, tiles etc.

**Information on usage:** The cuspidor cleaner AF is used undiluted. To open the safety cap, press the smooth surfaces together and twist. Wet the surface of the spittoon with approx. 20 ml of the cleaner and allow it to work in for a short period and spread it with a sponge or brush. After at least a minute working in, rinse with water. It is recommended to use after every soiling.

**Spectrum of action:** Bactericidal and fungicidal (Candida albicans) according to EN 13697, tuberculocidal according to EN 14348.

**Contact time:** Bacteria and Candida albicans according to EN 13697: Undiluted: 1 min. (low load)  
Tuberculosis inactivation (M. terrae) according to EN 14348: Undiluted: 5 min. (low load)

**Composition:** 100 g contains: 1.0 g bis(3-Aminopropyl)dodecylamine, 25.0 g ethanol.

**Safety precautions:** H226 - Flammable liquid and vapour. H315 - Causes skin irritation. H319 - Causes serious eye irritation. Keep away from heat/sparks/open flames/hot surfaces. - No smoking. Wear protective gloves and eye protection. IF ON SKIN: Wash with plenty of soap and water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice / attention. If eye irritation persists: Get medical advice / attention. Store in a well-ventilated place. Keep cool. Dispose of contents/container in accordance with local/regional/national/international regulations.

Keep container tightly closed. Only collect completely empty packages for recycling. Dispose of residual filling material under EAK 07 06 99 (European Disposal Catalogue).

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