TRAY CLEANER

Stand 01/2018

REF 100 701

Inhalt/Content/Contenu/Zawartość/Obsah/Tartalma:
e 5 Liter/ltr./litre/litr/liter

(Konzentrat/concentrate/Concentré/
koncentrat/koncentrát/Koncentrátum)

**Gefahr/Danger/
Niebezpieczeństw/
Nebezpečí/Veszély**

M+W Dental

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☉ The M+W tray cleaner is a concentrate for the careful cleaning of shaping spoons and instruments. The special cleaner quickly removes residues of alginates, zinc oxide eugenol, carboxylate- and phosphate cements and thermoplastic moulding compounds and waxes. The constituents are bio-degradable.

Use: The M+W tray cleaner is used at a concentration of 5-10%, and in the case of heavy soiling can be used as a pure concentrate. Add approx. 1 full dosing chamber per ltr. of warm water. The material residues are softened by the solution within a few hours, and can then be removed easily with a brush. The solution can also be used in an ultrasonic bath. The special cleaner is suitable stainless items. Chrome, nickel and aluminium parts should not be cleaned together. Danger of corrosion!

H318 Causes serious eye damage.

H315 Causes skin irritation.

H226 Flammable liquid and vapour.

Keep away from heat/sparks/open flames/hot surfaces. - No smoking. Keep container tightly closed.

Wear protective gloves and eye protection.

IF ON SKIN: Wash with plenty of soap and water.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Call a POISON CENTER or doctor/physician if you feel unwell.

IF SKIN IRRITATION OCCURS: Get medical advice / attention.

IN CASE OF FIRE: Use for extinction: CO₂, powder or water spray. Store in a well-ventilated place. Keep cool.

Dispose of contents/container in accordance with local/ regional/national/ international regulations.

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