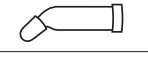
CONTENTS

| | |
|--|-----------------|
|  | 1. Unidose™ Tip |
|  | 2. Syringe |

| en - ENGLISH |
|--------------|
|--------------|

SimpliShade™ Bulk Fill

Nanohybrid Restorative Composite
ISO 4049: Polymer-Based Dental Restorative Material, Class 2/Group 1

GENERAL INFORMATION

SimpliShade Bulk Fill is an intra-oral, light-cured, esthetic, resin-based nanohybrid dental restorative that is designed for the placement of direct restorations. SimpliShade Bulk Fill is OptiBond™ Family, a product from the Kerr OptiBond™ Family.

SimpliShade Bulk Fill is available in Unidose™ compules and syringe delivery systems that are suitable for occlusal surfaces. The product allows a technique in which a cavity up to 5 mm in depth can be filled and cured in a single increment.

SimpliShade Bulk Fill is offered in one universal shade to optically blend with a range of VITA™ shade guide equivalents.

This composite can be polished with a wide variety of polishing discs, rubber cups, and brushes.

When required per the European Medical Device Regulation (MDR EU 2017/745) a Summary of Safety and Clinical Performance Document (SSCP) is available and can be obtained at the following website: <https://ec.europa.eu/tools/edumad>

The Basic UDI-DI for this device is 805151187100011458.

Intended Purpose: The intended purpose of SimpliShade Bulk Fill is for restoration of teeth as a core build-up material.

Intended users: This product is to be used by qualified dental professionals.

Intended Patient Population: The device is for use in a general patient population in need of a restoration due to missing tooth structure.

Clinical Benefit: it restores the anatomy of the tooth and its function.

PRODUCT COMPOSITION

SimpliShade Bulk Fill composite is radiopaque with about 250% aluminum (100% aluminum has a radio-opacity equivalent to that of dentin and 200% aluminum is equivalent to enamel).

The fillers are comprised of chemically infused mixed oxides, pre-polymerized filler, barium glass filler, silica, and ytterbium trifluoride (total filler loading of approximately 82.5% by weight or 66% by volume).

The inorganic filler content is up to 75% by weight or 55% by volume. SimpliShade Bulk Fill

| | | | |
|--|---|---|---|
|  |  |  |  |
|  |  |  |  |
|  |  |  |  |
|  |  |  |  |
|  |  |  |  |

| |
|--|
|  |
|  |
|  |
|  |
|  |
|  |
|  |
|  |
|  |
|  |
|  |

Instructions For Use
DMC4362_Rev.00 2024-01
Kerr Italia S.r.l.
Via Pissarello, 174, Scaldasola (CA), 06018 Italy
+39 081 850 6311 | kerr@kerr.com
UK
4 Lonswalk Road, Stockley Park
Uxbridge UB 11 1TF, United Kingdom
27975
CE
+91 8870 3000
+1 8870 3000
©2024 Kerr Corporation.
All trademarks are property of Kerr Corporation. All Rights Reserved.

has a primary particle size range of 40 nm to 0.4 µm. The product contains Bis-EMA, Bis-GMA, UDMA, and TEGDMA resins in an approximate concentration of 14% to 37% by weight. The product also contains quinones, phenols, polyesters, polyamines, benzoates, and inorganic oxides in a concentration of less than 1% each.

None of the ingredients in the product contain pharmacologically active components.

INDICATIONS FOR USE

SimpliShade Bulk Fill is indicated for direct placement of anterior and posterior restorations, core build-ups, and repair of defects in restorations.

CONTRAINDICATIONS

None known.

WARNINGS

- Do not use this product in patients with known methacrylate allergies. The dental professional should inform the patient that this product is known to cause a methacrylate allergic reaction.
- Avoid product application in close proximity to the pulp tissue, since exposure to this product may injure the pulp.
- Avoid skin, eye, and soft tissue exposure to this product, since uncured methacrylate may cause an irritation reaction. Wash thoroughly with water after contact.
- In case of contact with eyes, rise immediately with water and seek medical attention.
- Do not eat internally.

PRECAUTIONS

- Provide the patient with protective eyewear.
- Use a rubber dam to isolate the oral cavity.
- Avoid contamination of the preparation with fluids during restorative placement.
- Use of protective gloves and a no-touch clinical technique is strongly recommended.
- If the composite is touched with commonly used gloves, acrylate may penetrate the gloves. Dispose of gloves immediately and wash hands.
- Do not use the restorative material on surfaces that are contaminated with hydrogen peroxide (H₂O₂) or eugenol, as its residues can adversely affect the polymerization of the composite.

- Protect the material from ambient light that may initiate polymerization.
- Do not open the Unidose composite foil package until initial use. Use the product within 6 months of opening the pouch, but always ensure that the composite is not past the expiration date printed on the label.
- The Unidose tips are designed for SINGLE PATIENT USE ONLY to prevent cross-contamination between patients.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the legal manufacturer, Kerr Italia, S.r.l., and to the competent authority for that country in which the user/ or patient is established.

STEP-BY-STEP INSTRUCTIONS

Limitations of Use

- When a dry working field cannot be established.
- The step-by step working instructions cannot be applied.
- The restoration cannot be properly polymerized.

Notes:

- Wear personal protective equipment when using and handling this product.
- Do not open the Unidose composite foil package until initial use. Use the product within 6 months of opening the pouch.
- Isolate the tooth with your preferred isolation method.
- Prepare the tooth using your preferred rotary instruments and using contemporary preparation designs. SimpliShade Bulk Fill composite offers excellent blending (chameleon effect) which may be further enhanced by the use of small bevels on preparation margins. If replacing a restoration, be sure to remove all existing restorative material from the operative site.
- If a pulp exposure has occurred, place a small amount of calcium hydroxide followed by the placement of a conventional glass ionomer base.
- SimpliShade Bulk Fill is compatible with all methacrylate dental adhesive systems. Complete the bonding procedure per manufacturer's instructions.
- If desired, place a liner using a low viscosity flowable composite and light cure according to manufacturer's instructions.
- In situations of discoloration of the tooth structure, a universal opaque composite shade can be used to block out the discoloration or hide a metal portion if it exists.
- SimpliShade Bulk Fill features optimal, non-sticky handling. It permits the use of hand instruments to define most of the restoration anatomy, which will help minimize the time and effort associated with rotary instrumentation.

- For **Unidose delivery:** Inject composite material from the Unidose tip directly into the cavity, starting with the deepest portion.
- For **Syringe delivery:** Extrude the material from the syringe prior to immediate use. After extruding material onto a pad, back off the syringe at least 2 full turns to relieve pressure on the material.

- For Class I, Class II, and core-build-ups, apply the composite up to a 5 mm* increment, taking care to properly adapt the material to the cavity surface.
- Light cure the composite*. Refer to the table at the end of these instructions for recommended curing times of the various shades at various light intensities.
- Adjust occlusion as needed. Complete the finishing and polishing using your preferred polishing system.

*RECOMMENDED CURING TIMES

Adequate curing is essential for restorative function. Uncured or undercured composite material is far more likely to experience some form of clinical failure. Kerr recommends regular inspections of your curing lights, to ensure they are functioning and providing the expected irradiance. SimpliShade Bulk Fill is intended to be cured by a Halogen light with a minimum light intensity of 650 mW/cm², or an LED light with a minimum light intensity of 1,000 mW/cm² and a light wavelength output within the 400-520 nm range. Please refer to the table below for curing recommendations. All times refer to a single occlusal surface cure. In any posterior restoration, Kerr recommends additional 10 second cures on both the buccal and lingual/palatal surfaces of the tooth.

For all other lights, see the manufacturer's recommendations.

| Curing Method | Universal Shade |
|--|-----------------|
| Incremental depth | Up to 5 mm |
| Optilux™ (or light with output 650-1,000 mW/cm ²) | 20 seconds |
| Demi™ Ultra/Demi™ Plus (or light with output >1,000 mW/cm ²) | 10 seconds |

CLEANING AND DISINFECTION

Unidose tips: To prevent the cross-contamination between patients, do not re-use Unidose tips.

Unidose dispensing guns: Refer to the reprocessing instructions for the manufacturer of the dispensing gun you are using.

Composite syringes: To prevent syringes from getting contaminated with saliva or other bodily fluids, use protective barriers. Always handle the syringe with clean/disinfected gloves. Do not re-use syringe if it becomes contaminated.

Always replace the syringe cap after each use, to prevent the resin from hardening in the syringe.

SHELF LIFE, STORAGE AND DISPOSAL

SimpliShade Bulk Fill should be stored at temperatures between 2°C to 28°C (36°F to 82°F). Avoid excess heat. If the product is stored in a refrigerator, allow the product to reach room temperature prior to use.

The expiration date is marked on the packaging. Do not use the restorative after the expiration date. Once the foil pouch is opened, used within 6 months and prior to the expiration date printed on the label.

For proper disposal always follow federal or state laws applicable to that country.

LIMITED WARRANTY - LIMITATION OF KERR ITALIA'S LIABILITY

Kerr Italia's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr Italia's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr Italia's product. In the event of a defect in material or workmanship, Kerr Italia's liability is limited, at Kerr Italia's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr Italia. In no event shall Kerr Italia be liable for any indirect, incidental, or consequential damages.

EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR ITALIA, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

Kerr, SimpliShade, OptiBond, and Unidose are trademarks owned by Kerr Corporation.

©2024 Kerr Corporation. All trademarks are property of Kerr Corporation. All Rights Reserved. VITA is a trademark of VITA Zahnfabrik Company, Germany.

Full explanation of symbols used on Kerr packaging is located at:

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Kerr, SimpliShade, OptiBond, and Unidose are trademarks owned by Kerr Corporation. ©2024 Kerr Corporation. All trademarks are property of Kerr Corporation. All Rights Reserved. VITA is a trademark of VITA Zahnfabrik Company, Germany.
Ker, SimpliShade, OptiBond, and Unidose are trademarks owned by Kerr Corporation. ©2024 Kerr Corporation. All trademarks are property of Kerr Corporation. All Rights Reserved. VITA is a trademark of VITA Zahnfabrik Company, Germany.