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## EN MD 530 Cement remover and denture cleaner

MD 530 is a special cleaner for the removal of acid-soluble stains, such as cements and can also be universally used as a denture cleaner. MD 530 is mainly characterised by its intensive cleaning action and very good corrosive-inhibiting properties. MD 530 easily cleans prostheses, bridges, crowns, spatulas, glass, etc. and removes cement, calculus, oxidation layers, tobacco stains, etc. In addition, corrosion and flux residues can be eliminated. **Composition:** MD 530 contains inorganic acids (5 - 15 %), surfactants (< 1 %), and auxiliary substances.

**Instructions for use:** MD 530 is used undiluted at room temperature. Place the parts to be cleaned in MD 530 and allow a maximum of ten minutes for it to take effect. When using an **ultrasonic cleaner**, do not apply the solu-

tion directly into the tank, pour it into a container insert. We recommend an exposure time of two to a **maximum** of four minutes for instruments made of stainless steel, and up to ten minutes for all other parts made of glass, porcelain or plastic. After treatment, thoroughly rinse with water and allow to dry. Renew solution when necessary. For further information please also see product information on the Internet.

**Caution:** MD 530 may only be used for instruments and objects made of acid-resistant metal, glass, porcelain or plastic. Do **not** put general and rotary instrumentation, with the exception of spatulas and stoppers, in MD 530. MD 530 is **not** suitable for light and non-ferrous heavy metal as well as non-stainless steel (e.g. burs, milling tools and root canal instruments). Changes in the colour and odour of MD 530 during storage have no impact on efficacy. Use exclusively by qualified personnel. All serious events that occur in association with the product must be immediately reported to the manufacturer and the competent authority of your member state.

  ☞ see imprint.

Warning. Causes serious eye irritation. Wear protective gloves/eye protection/face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. Store in a well-ventilated place. Keep container tightly closed. Dispose of contents/container to hazardous or special waste collection point.

UK-REP: DÜRR DENTAL (Products) UK Ltd., 14 Linnell Way - Telford Way Industrial Estate, Kettering, Northants NN16 8PS, United Kingdom, info@duerruk.com

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