DE MD 550 Mundspülbecken-Reiniger

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EN MD 550 Spittoon bowl cleaner

MD 550 is a special, ready-to-use, antibacterial preparation that cleans and cares for spit-

toon bowls of dental units. MD 550 removes limescale deposits, blood, residues of plaque staining tablets, impression materials, etc.

Composition: 100 g MD 550 contains 40 g Ethanol, < 1 g surfactants, < 5 g organic acids, auxiliary agents, water, benzyl salicylate.

Effective spectrum: Bactericidal, yeasticial. Tested in accordance with EN 13727, EN 13624.

Directions for use: MD 550 is a ready-to-use solution. To open the safety cap, press the smooth surfaces together and turn. Squirt a little MD 550 into spittoon bowl and spread over the surface with a sponge applying even pressure. Allow a few minutes for preparaion to take effect, then rinse with water. Apply MD 550 as required, however, in the evening it must be done before disinfecting the suction system.

Caution: Flash point 23 °C. Take care with alcohol-sensitive materials (e.g. acrylic glass).

 ☞ see imprint.

Warning. Highly flammable liquid and vapour. Keep away from heat/sparks/open flames/hot surfaces. - No smoking. Wear protective gloves. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Store in a well-ventilated place. Keep cool. Dispose of contents/container to hazardous or special waste collection point.

DÜRR DENTAL (Products) UK Ltd. 14 Linnwell Way – Telford Way Industrial Estate Kettering, Northants NN16 8PS, United Kingdom ☎ +44 1536526740

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