OptiBond™

Gel Etchant

Phosphoric Acid 37.5%

Instructions For Use

Kerr™

SYMBOLS DESCRIPTIONFull explanation of symbols used on Kerr packaging located at: <http://www.kerrdental.com/symbols-glossary>

	Catalogue number
	Manufacturer
	Batch code
	European conformity
	Consult an electronic instructions for use https://www.kerrdental.com/download-center
	Consult Instructions for Use
	Caution
	Authorized representative in the European Community
	Use-by date
	Do not re-use
	Temperature Storage Limitations: Low 2°C/36°F and High 25°C/77°F
	Medical Device

CONTENTS

	1. Gel Etchant Syringe (3 g)
	2. Gel Etchant Tip
	3. Gel Etchant Syringe (30 g)
	4. Luer Connector for Intraoral Syringes Filling

en - ENGLISH

OPTIBOND™ GEL ETCHANT

Phosphoric Acid 37.5%

GENERAL INFORMATION

Kerr's distinctive purple-colored 37.5% phosphoric acid gel is designed for maximum etching efficiency. Its optimal viscosity allows the material to stay where it is placed, yet can be rinsed off easily.

PRODUCT COMPOSITION

Phosphoric Acid 37.5%

INDICATIONS FOR USE

OptiBond Gel Etchant is designed for dentin/enamel etching to prepare the tooth surface prior to the application of a bonding agent.

CONTRAINDICATIONS

None Known.

WARNINGS

1. OptiBond Gel Etchant contains 37.5% phosphoric acid. Avoid contact with skin, eyes, and soft tissue. Wash thoroughly with water after contact. Get medical attention for eyes. Do not take internally.
2. Avoid product application in direct contact with pulp tissue since exposure to this product may injure the pulp.

PRECAUTIONS

1. This product is intended to be used by qualified dental professionals on a general patient population.
2. Use personal protective equipment when using and handling this product.
3. Use a rubber dam to isolate the treated tooth and to protect the surrounding soft tissues.
4. Gel Etchant dispensing tips are designed for single use only, to prevent cross contamination between patients.
5. Intraoral 3g syringes, protective caps, and a connector cannot be sterilized.
6. Never use excessive force on syringe plunger to dispense etchant which may lead to over dispensing of the material. If resistance is felt replace current dispensing tip with a new tip and check flow outside the oral cavity.
7. Replace the syringe cap after each use to prevent the product from drying in the syringe.
8. Do not fill etchant from small intraoral syringe back to Economy syringe.

ADVERSE EVENTS

If a serious incident occurs with this medical device, report it to the manufacturer and to the competent authority for that country in which the user and/or patient is established.

STEP-BY-STEP INSTRUCTIONS**Etching procedure of a tooth surface**

1. Use a rubber dam to isolate the oral cavity.
2. Pumice clean unprepared tooth structure with a fluoride-free cleaning paste; rinse thoroughly with water spray and air dry.
3. Securely attach the dispensing tip to the OptiBond Gel Etchant intraoral syringe and check the flow extraorally.
4. Apply OptiBond Gel Etchant to enamel and dentin surfaces for 15 seconds.
5. Rinse thoroughly for 10 seconds with water or until etchant has been completely removed.
6. Gently air dry for 5 seconds being careful not to desiccate dentin.
7. Proceed with application of a bonding agent per manufacturer's instructions.

Refilling of empty 3g intraoral syringe

1. Unscrew a protective cap from 30g Economy syringe, as well as from an empty 3g intraoral syringe.
2. Screw the connector on the Economy syringe on one side and on the empty intraoral syringe on the other side.
3. Ensure that the two syringes are perfectly tightened to the connector. Do not strip the thread. Fill the empty syringe by slowly and steadily pulling the plunger of the small syringe. Simultaneously, exert light pressure on the plunger of the Economy syringe using the other hand to support the refilling process.
Note: Do not point syringes toward people during refilling. Do not overcharge the small syringe.
4. Disconnect the two syringes by an anticlockwise twist.
5. Replace the protective cap on the Economy syringe.
6. Attach the dispensing tip to the small syringe. The syringe is now ready for use.
7. To ensure complete product traceability, small syringes can be filled only using the product contained in the Economy syringe provided in the same package. Apply on small syringes the two white labels provided in the package, transcribing lot number and expiry date from the Economy syringe.

CLEANING AND DISINFECTION

To prevent cross contamination, use disposable plastic syringe sleeve protection cover or protective barrier wrap. Always handle the syringe with clean gloves.

Do not reuse if the syringe becomes contaminated.

STORAGE AND DISPOSAL

Product should be stored between 2°C to 25°C (36°F to 77°F). If product is stored in a refrigerator allow the product to reach room temperature prior to use. The expiration date is marked on packaging. Do not use after expiration date. For proper disposal always follow local and national regulations.

The SDS for this product is available at: kerrdental.com. It can also be obtained from your usual supplier.

LIMITED WARRANTY - LIMITATION OF KERR'S LIABILITY

Kerr's technical advice, whether verbal or in writing, is designed to assist dentists in using Kerr's product. The dentist assumes all risk and liability for damages arising out of the improper use of Kerr's product. In the event of a defect in material or workmanship, Kerr's liability is limited, at Kerr's option, to replacement of the defective product or part thereof, or reimbursement of the actual cost of the defective product. In order to take advantage of this limited warranty, the defective product must be returned to Kerr. In no event shall Kerr be liable for any indirect, incidental, or consequential damages. EXCEPT AS EXPRESSLY PROVIDED ABOVE, THERE ARE NO WARRANTIES, BY KERR, EXPRESS OR IMPLIED, INCLUDING WARRANTIES WITH RESPECT TO DESCRIPTION, QUALITY, OR FITNESS FOR A PARTICULAR PURPOSE.

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Kerr Corporation
Kerr Italia S.r.l.
Ormco B.V.