ProBase® Hot

[en] Instructions for Use

Heat-curing denture base material

Rx ONLY

CE 0123

Manufacturer:
Ivoclar Vivadent AG
Bendererstrasse 2
9494 Schaan/Liechtenstein
www.ivoclar.com

Complies with ISO 20795-1;
EN ISO 20795-1

Date information prepared:
2023-01-26 / Rev. 0

ivoclar

English

1 Intended Use

Intended purpose

- Fabrication of bases for removable dentures
- Fabrication of hard occlusal splints
- Fabrication of orthodontic appliances

Patient target group

- Patients with permanent teeth
- Adult patients with dental implants
- Edentulous adult patients

Intended users / Special training

- Dental laboratory technicians (fabrication of restorations in the dental laboratory)
 - Denturists (fabrication of restorations in the dental laboratory)
- No special training required.

Use

For dental use only.

Description

Heat-curing denture base material for the packing technique.

Technical data

| Characteristics | Value |
|---------------------------|--------------------------|
| Flexural strength | ≥ 65 MPa |
| Flexural modulus | ≥ 2000 MPa |
| Residual MMA content | ≤ 2.2 % |
| Water absorption (7 days) | ≤ 32 µg/mm ³ |
| Solubility (7 days) | ≤ 1.6 µg/mm ³ |

Indications

- Partial edentulism in the anterior and posterior region
- Complete edentulism
- Dental dysfunctions and parafunctions
- Teeth misalignments

Areas of application

- Complete denture prosthetics
- Partial denture prosthetics
- Implant prosthetics
- Occlusal splints
- Orthodontic appliances

Contraindications

The use of the product is contraindicated if the patient is known to be allergic to any of its ingredients.

Limitations of use

- Direct contact of uncured material with intraoral tissues.
- The ProBase Hot/Cold shade guide cannot be sterilized and is thus not suitable for use on the patient.
- High heat generation (exceeding 110 °C / 230 °F) should be avoided during the grinding, polishing and cleaning process. Avoid contact with solvents.

Processing restrictions

The product is not intended to be reprocessed or reused.

Side effects

In individual cases, local allergic reactions to polymethyl methacrylate materials have been reported.

Interactions

There are no known interactions to date.

Clinical benefit

- Reconstruction of chewing function
- Restoration of esthetics

Composition

- **ProBase Hot polymer**
Polymethyl methacrylate, tributyl o-acetyl citrate, dibenzoyl peroxide
- **ProBase Hot monomer**
Methyl methacrylate, ethylene glycole dimethacrylate

2 Application

Application notes

- Make sure that the precast and model have been soaked well in water.
- Apply two layers of the alginate-based isolation agent "Separating Fluid" from Ivoclar and allow to dry thoroughly.
- Roughen the teeth well and wet them with monomer to ensure a sound bond with the denture base.
- The denture base material must be thoroughly mixed using a hard spatula (approx. 20-30 seconds).
- Avoid fast cooling in cold water (tension cracks).
- In order to select the appropriate shade of ProBase Hot, the ProBase Hot/Cold shade guide is available to the user.
- For test pressing and to check the amount of material used, the isolating foils included in the delivery can be used.
- We do not recommend using a gloss varnish.
- The products are intended to be used only by professionals.
- Avoid mixing of different materials.

1. Preparation

Isolate boiled-out, water-saturated plaster surfaces with two layers of the alginate-based isolation agent "Separating Fluid" and allow to dry thoroughly. Roughen the teeth well and wet them with monomer to ensure a sound bond with the denture base.

2. Dosage

Ideal mixing ratio for one denture: 22.5 g polymer : 10 ml monomer

3. Mixing

Mix together powder and liquid with a spatula in the given mixing ratio and blend thoroughly for 20-30 seconds. Subsequently, leave the material to mature in the closed mixing cup at room temperature (23 °C / 73 °F) for 8 to 10 minutes. For this procedure, a mixing cup and mixing spatula are provided.

4. Working time

As soon as the material has matured sufficiently and is no longer sticky, it can be worked for approx. 20 minutes at 23 °C / 73 °F. Dough time and working time depend on the room temperature.

5. Pressing

Slightly overfill the hand-warm (approx. 40 °C / 104 °F) and isolated flask halves with resin dough. Carefully close the flask, load with 80 bar pressure and fix with a clamp. Maintain pressure.

6. Polymerization

Heat-polymerization can be carried out in different ways:

– Standard procedure

Place the closed flask in cold water. Heat up to 100 °C / 212 °F and let boil for 45 minutes.

– Alternative methods:

– Place the flask in cold water, heat up to 70 °C / 158 °F and leave it for 30 minutes. Then heat up to 100 °C / 212 °F and let boil for 30 minutes.

– Place the flask in boiling water. Bring the water to the boil again and then let boil for 40 minutes. This procedure is only suitable for small to medium-sized dentures.

With the ideal mixing ratio and the recommended procedure, a residual monomer content of $\leq 2.2\%$ is achieved.

7. Cooling

Let the flask cool at room temperature for 30 minutes. Subsequently, completely cool the flask with cold water.

8. Deflasking and finishing

Open the completely cooled flask and remove the plaster.

Subsequently, check the occlusion and finish and polish in the usual manner.

Repair and adjustment possibilities with ProBase Hot

Repairs and adjustments of ProBase Hot can be carried out by using ProBase Cold in the pouring technique. 

Thoroughly roughen the corresponding surfaces and wet with monomer.

3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclar.com, and your responsible competent authority.
- The current Instructions for Use are available in the download section of the Ivoclar Vivadent AG website (www.ivoclar.com).

Warnings

- Observe the Safety Data Sheet (SDS) (available in the download section of the Ivoclar Vivadent AG website www.ivoclar.com).
- The monomer contains methyl methacrylate (MMA)
- Methacrylate is easily flammable and irritating (flash point +10 °C / 50 °F).
- Irritating to eyes, skin and respiratory system.
- Wear protective goggles during processing.
- A mask should be worn and a suction device used during grinding.
- Skin contact may cause sensitization.
- Avoid contact of the skin with monomer or uncured material.
- Commercially available medical gloves do not offer protection against the sensitizing effect of methacrylates.
- Do not inhale vapours.
- Keep away from sources of ignition. Do not smoke.
- Do not empty into drains.
- Take precautionary measures against electrostatic discharge.

Disposal information

Remaining stocks must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks.

The following clinical residual risks are known:

- Ingestion of fragments
- Occurrence of denture stomatitis

4 Shelf life and storage

- Storage temperature 2–28°C (36–82 °F); we recommend storing monomers in the refrigerator.
- Do not use the product after the indicated expiry date.
- Expiry date: See note on the packages.
- Protect the product from direct sunlight.
- Before use, visually inspect the packaging and the product for damage. In case of any doubts, please contact Ivoclar Vivadent AG or your local dealer.

5 Additional information

Keep material out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the product for its suitability and use for any purpose not explicitly stated in the Instructions. The user is obliged to fully document and store the batch number, so that traceability of the batch is ensured.

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